


# User Technique Instructions.



HYPOGUARD

Remove one test strip from bottle and replace cap immediately.

1. Dip test strip in fresh, well-mixed urine sample for 1 second, making sure all the pads are wet.
2. Remove test strip from urine by drawing edge of strip along edge of urine sample cup.
3. Blot one side of the entire test strip on absorbent paper for 1 second to remove excess urine.

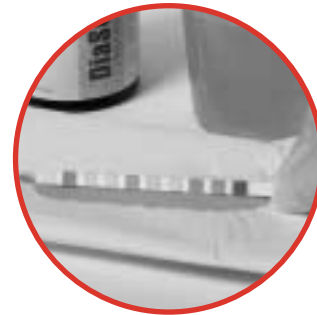

**Test Times:** All tests are read at **60 seconds**, except for the leukocyte test at **120 seconds**.

***IMPORTANT! Correct user technique helps ensure accurate results.***

## Questions?

Contact Customer Service at **800-818-8877** option 5 or **952-646-3200** option 5.

P/N 6097-00  
REV 11/02

DIASCREEN<sup>®</sup>