

HAEMOLANCE⁺ Plus[®]

Pediatric



A gentle and safe blood sampling device for heelsticks in premature infants, newborns and babies.



HYPOGUARD



HAEMOLANCE PLUS[®]

HAEMOLANCE⁺ Plus[®]

Pediatric



Features


- Single use, permanent lock-out blade helps prevent accidental needlesticks and cross-contamination
- Blade retracts after use
- Quick, dual spring action
- Protective blade cap
- Built in depth control assures uniform and accurate sampling results
- Ergonomic shape with gripping sides

Benefits

- Helps prevent accidental needlesticks and cross-contamination
- Extra safety
- Increased comfort
- Assures sterility and indicates usage
- Easy to use
- Easy to hold



1. Select the site and cleanse thoroughly.

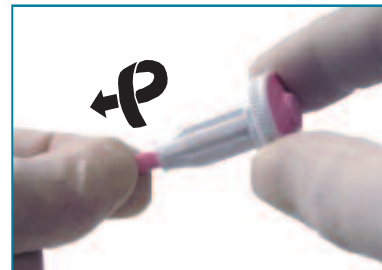

2. To remove the protective cover, turn and twist in one movement. Do not put thumb on the button while removing protective cap. This may cause a misfire.



3. Hold lancet between the fingers and place against the site, then press button firmly.



4. After use, discard in appropriate sharps container or according to local disposal ordinance.



Description	Product #	Depth	Width	Packaging
Haemolance Plus Pediatric (blade)	990400	1.20 mm	1.50 mm	150/Box

 **HYPOGUARD**
www.hypoguard.com